




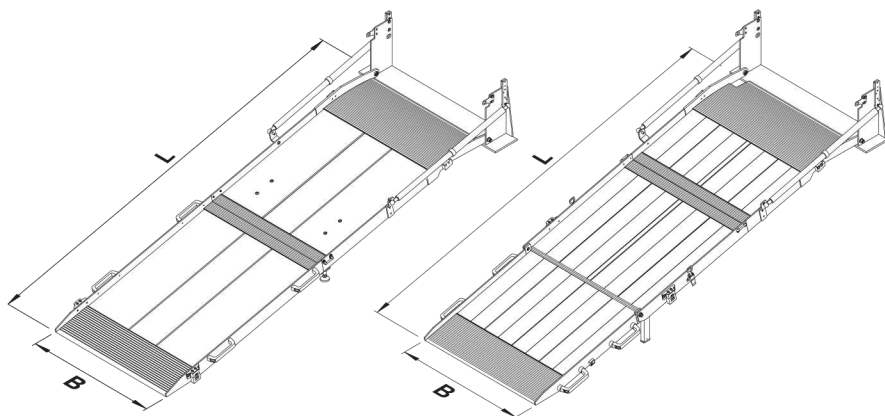
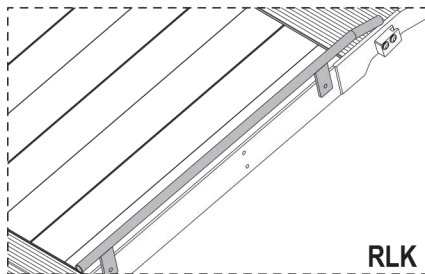
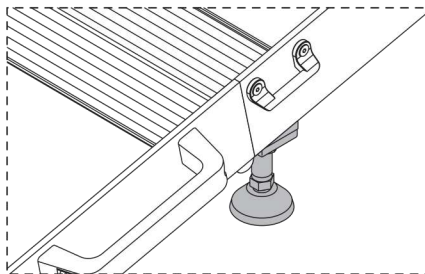
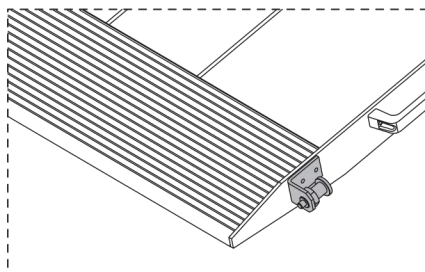
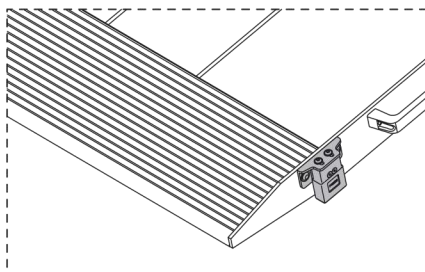
1 RLK, RRK










2 RLK-Z, RRK-Z

RLK

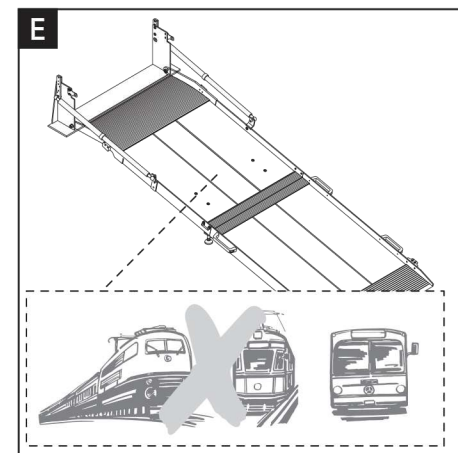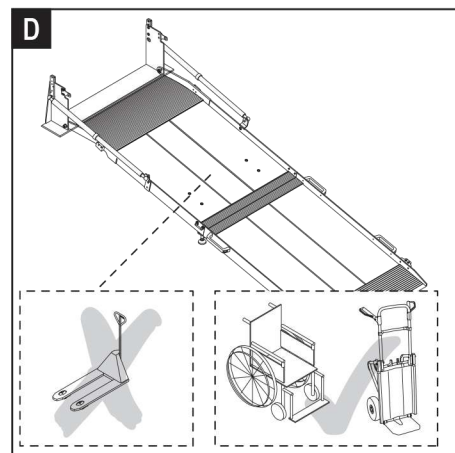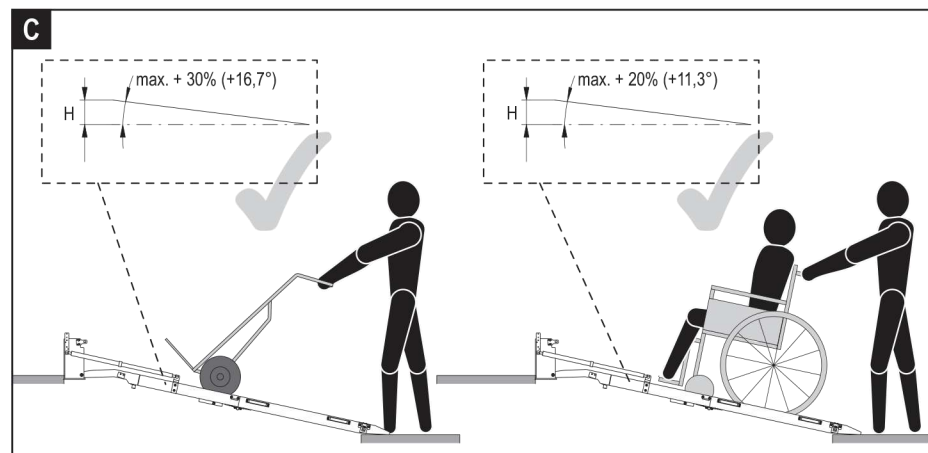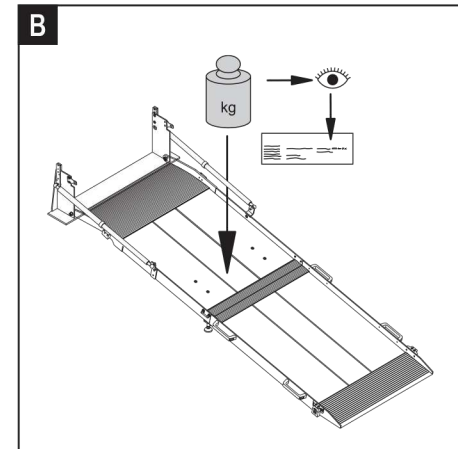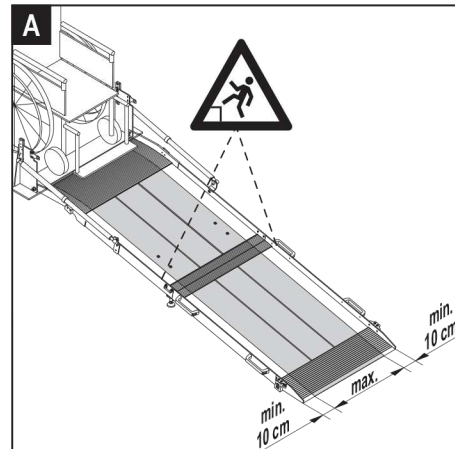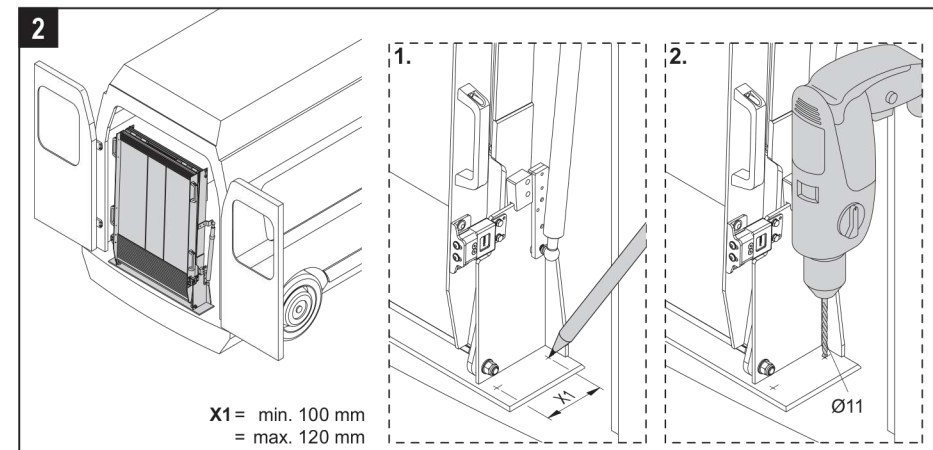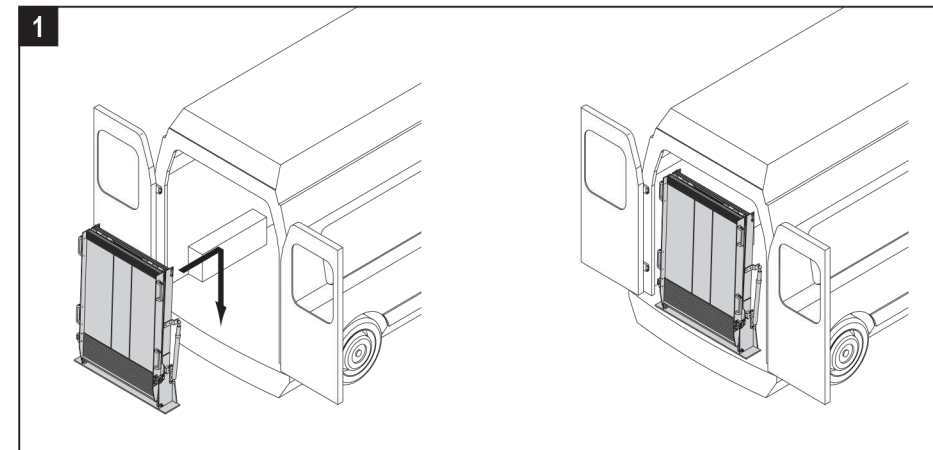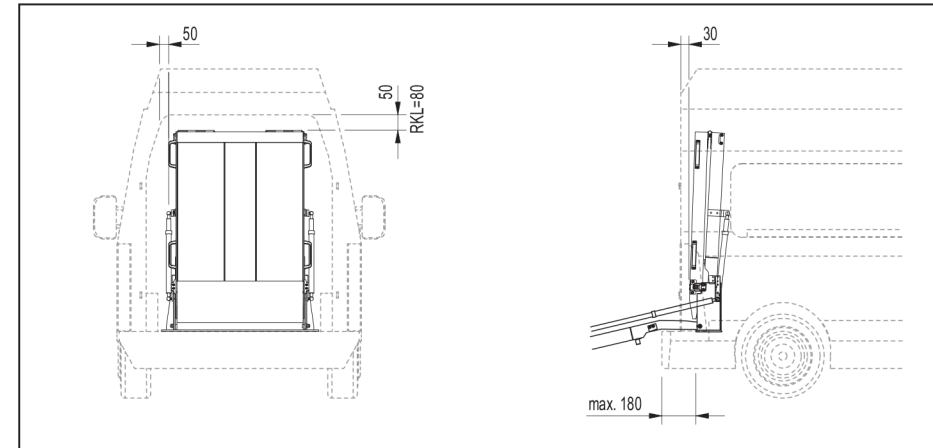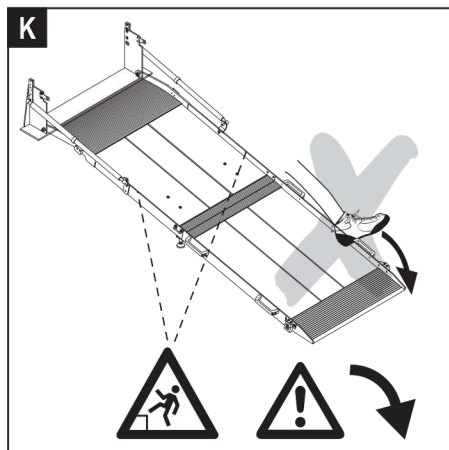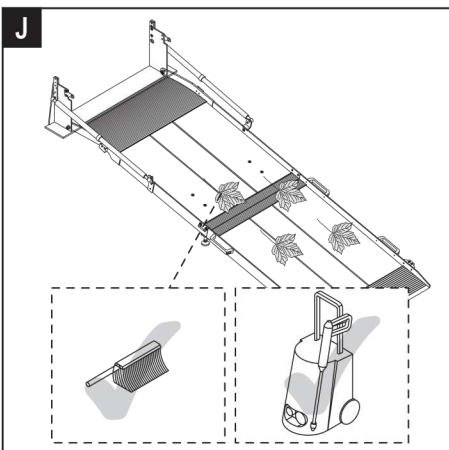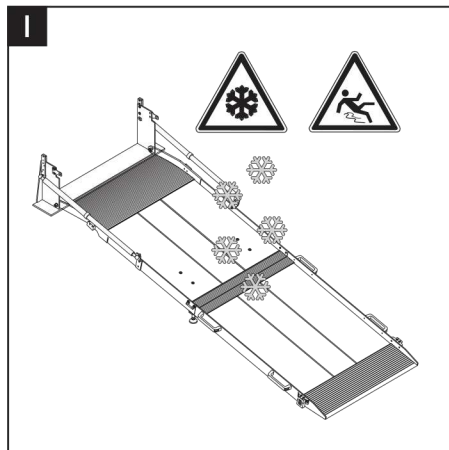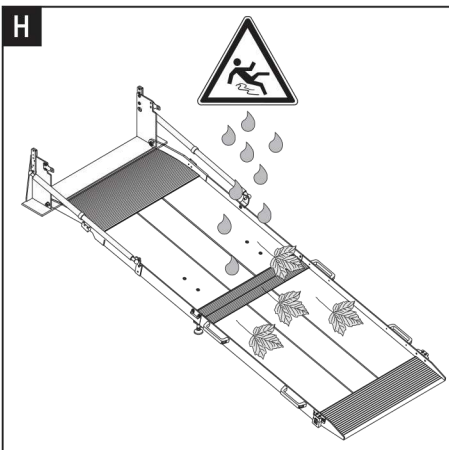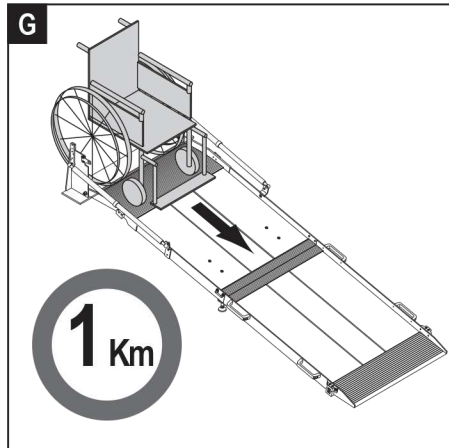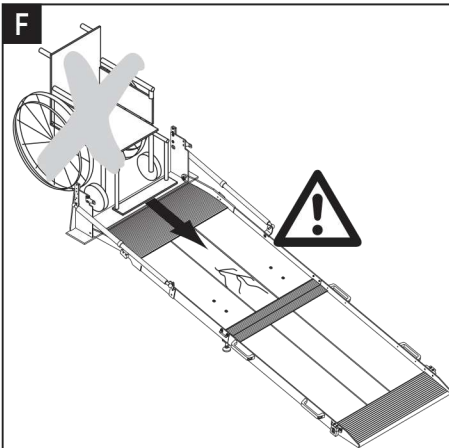
D, GB, FR, NL, ES, IT, PT, CZ, HU, RUS, PL, TR, SK, SL, SE, FI, NO, DK, GR, RO


3

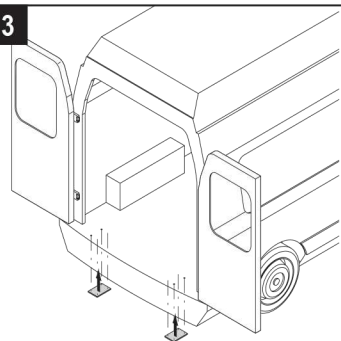
$X2 = \text{min. } 70 \text{ mm}$ $Y = \text{min. } 40 \text{ mm}$
 $= \text{max. } 120 \text{ mm}$ $= \text{max. } 80 \text{ mm}$

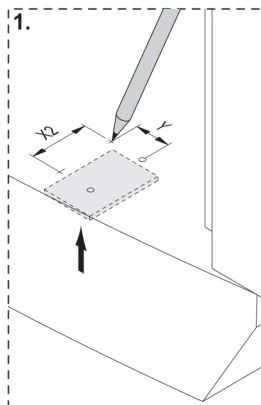
1.

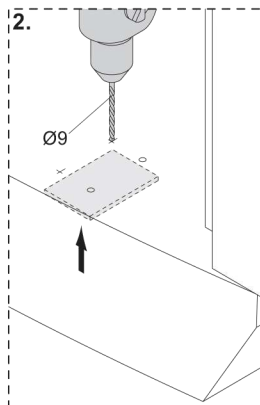
2.


4

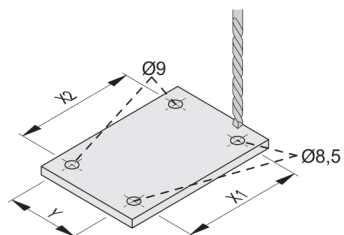
5

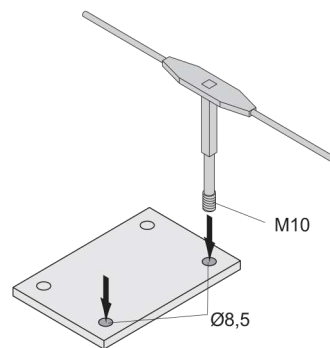
6

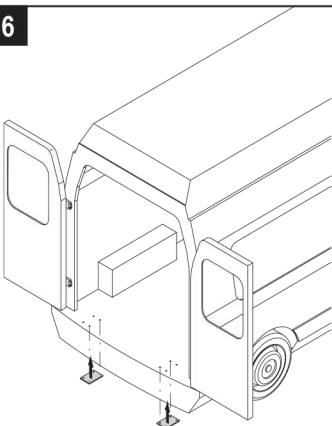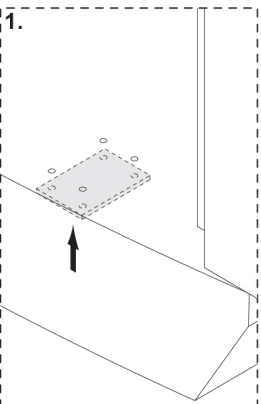
1.

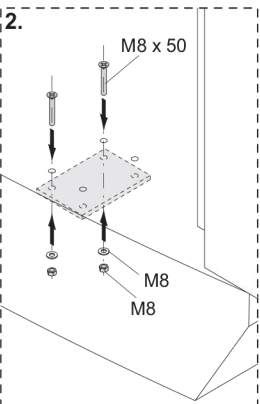
2.


6

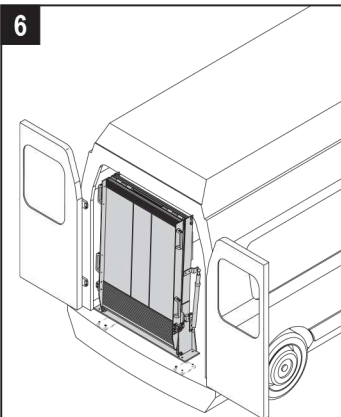
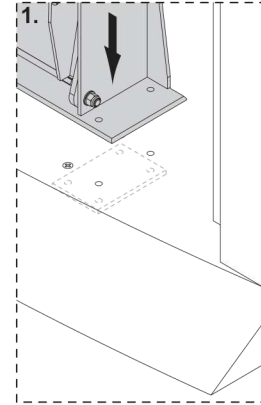
1.



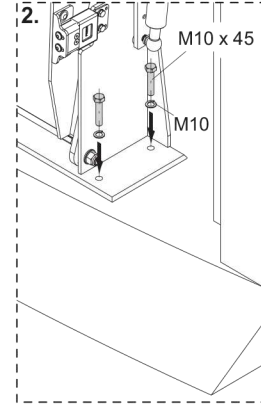
2.









7

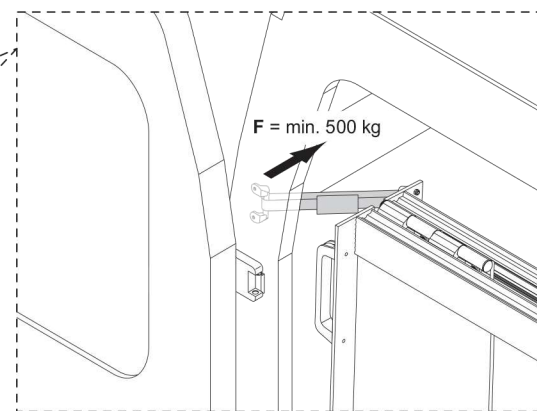
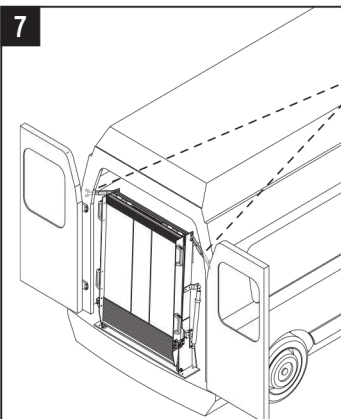
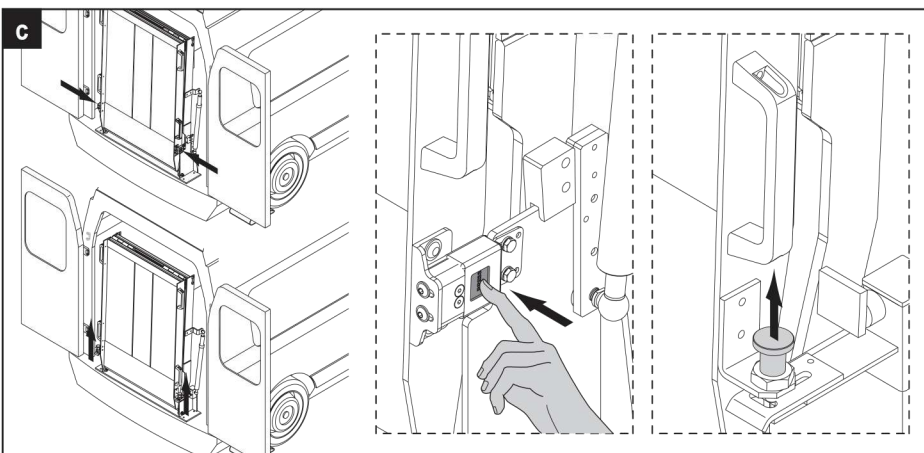
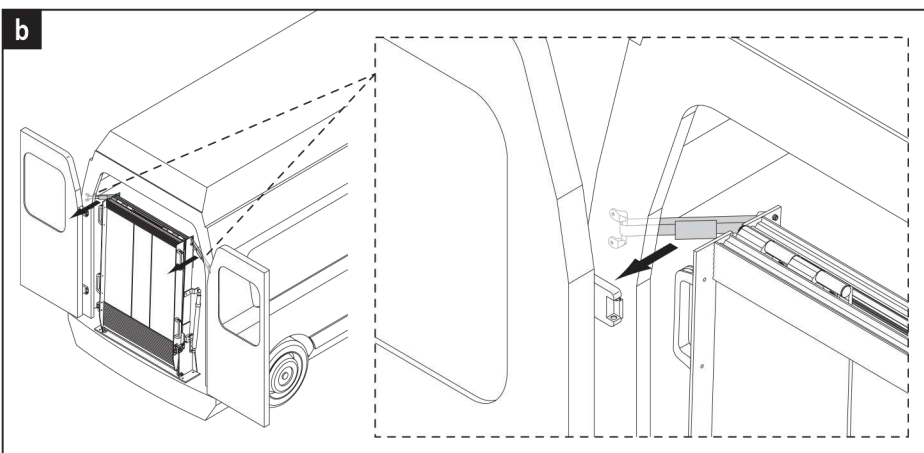
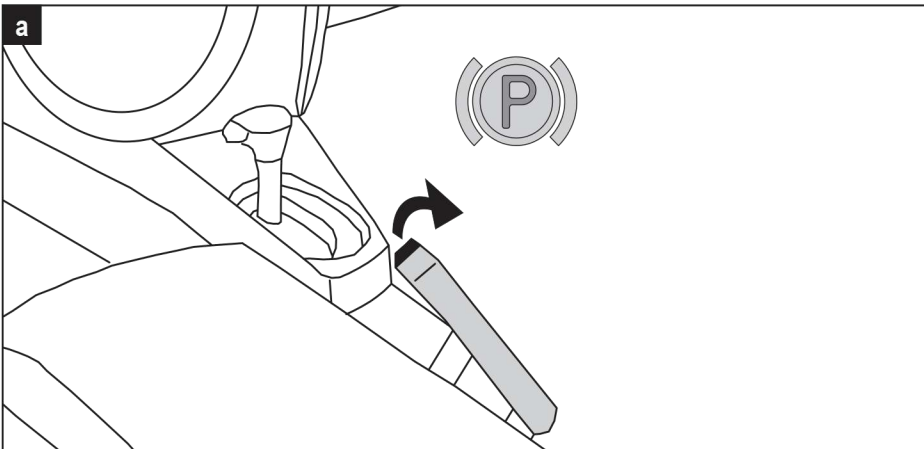
d | RLK, RRK

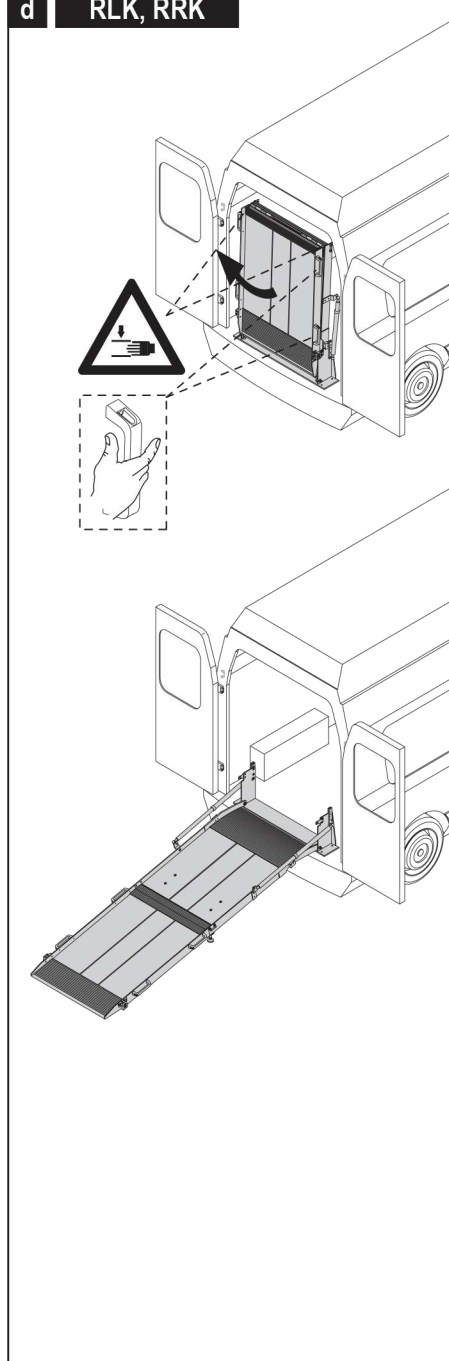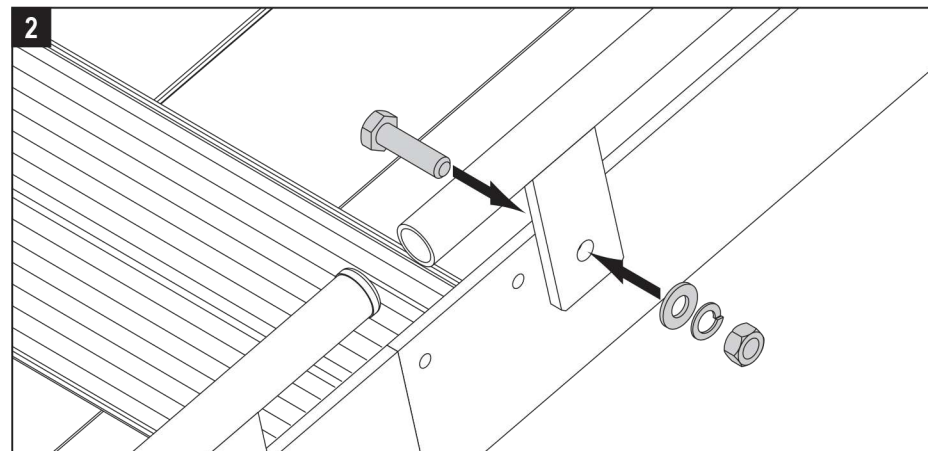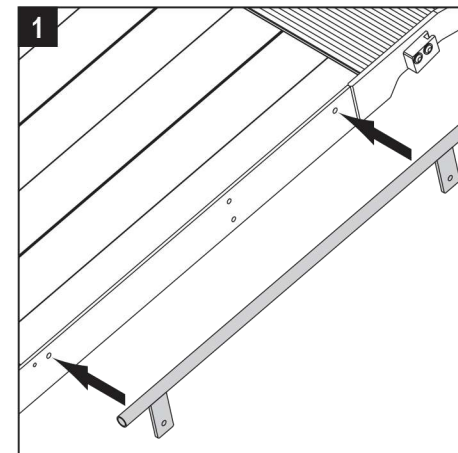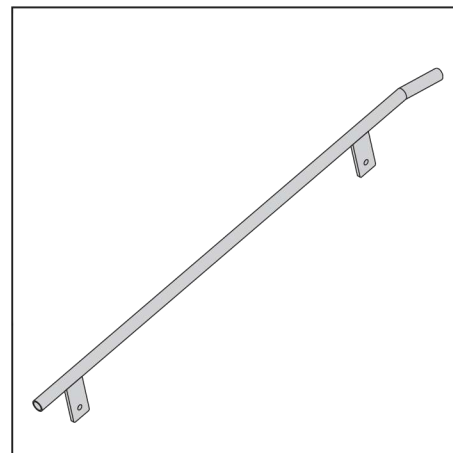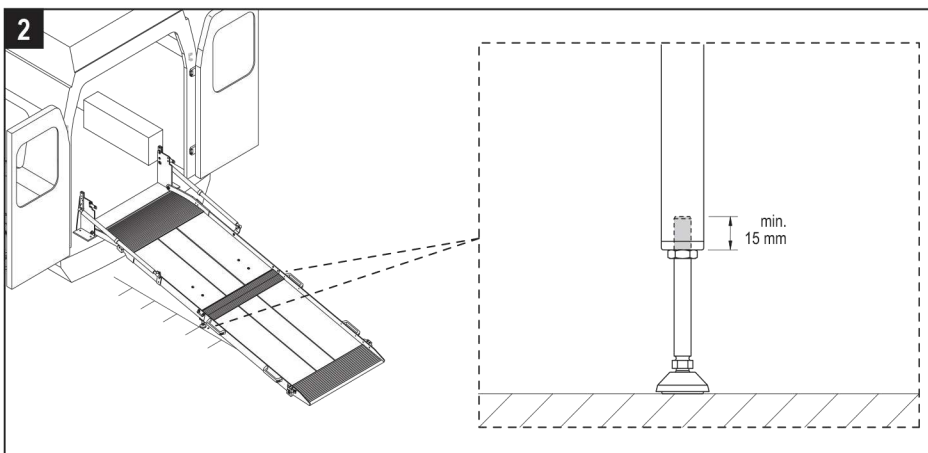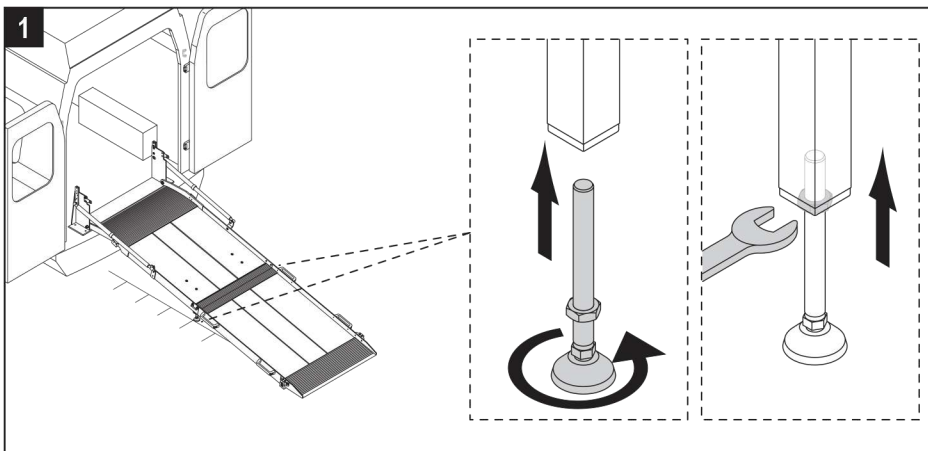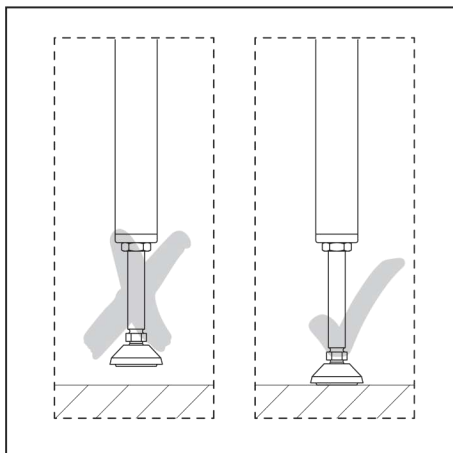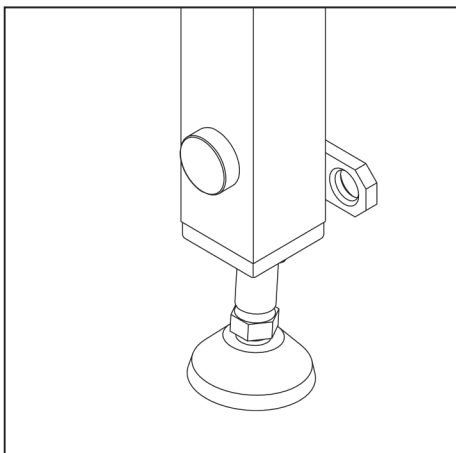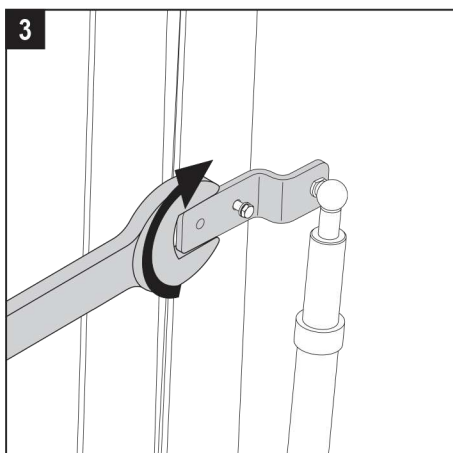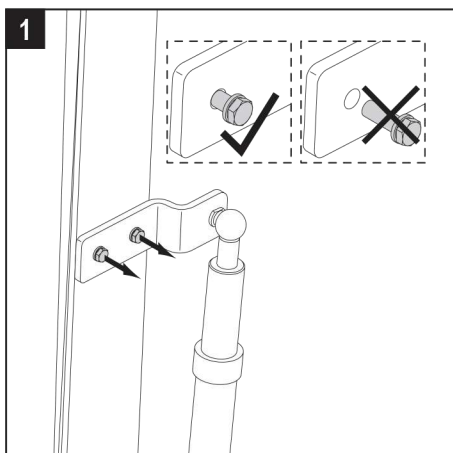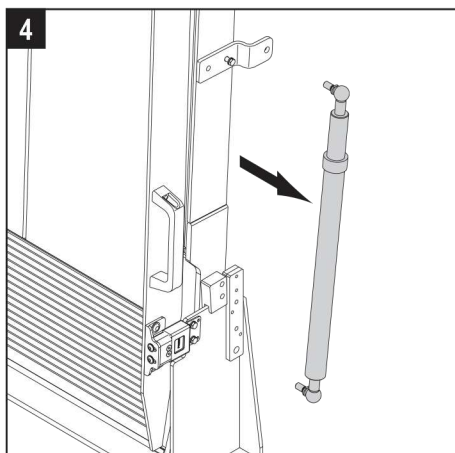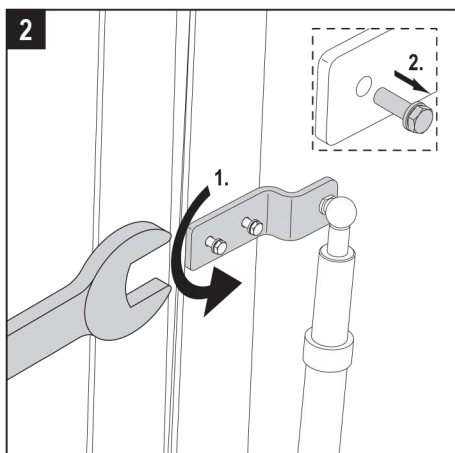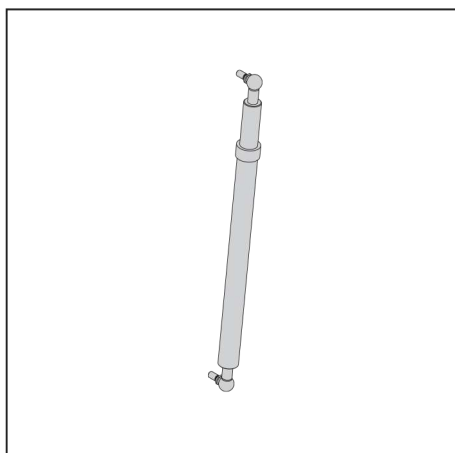D

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GB

Safety instructions

- Read the operating manual carefully. Only then may the product be utilised.
- Modifications to the product are subject to prior manufacturer approval.
- The product may only be used on stable terrain. The product must stand safely, i.e. it must not wobble.
- The product must be secured in such a manner that persons will not be endangered under normal traffic conditions.
- Loading and unloading of the vehicle must be carried out in such a manner that persons will not be endangered by objects falling or rolling down.
- The product when unfolded must be secured by the vehicle driver in such a manner that other road users (e.g. pedestrians, cyclists) will not be endangered.
- An incline >20% is not permissible in keeping with DIN 75078
- Follow the notices on the operation of electric wheelchairs
- Pneumatic spring under high pressure. Do not open!
- Do not heat above 80 °C.
- Impact or blows on the pneumatic spring will cause defects.

Check

Rule of thumb: an overloaded loading dock may not be used.

Before first commissioning

- Check all screw connections for proper seating
- Functional check
- Inspect pneumatic extension spring for damages.

Prior to each operation

- Visual check for any externally visible wear, tear and tear or damage (welding seams, drive surface)
- Completeness and full availability of all safety devices
- Inspect pneumatic extension spring for damages.

Monthly maintenance

- Check axial securing device of the hinge bolts for a firm seat.

Semi-annual maintenance

- Spray the hinges with a light machine oil.
- Visual check for any externally visible wear, tear and tear or damage (welding seams, drive surface)
- Functional check
- Check all screw connections for proper seating
- Clean any contamination from the running surface to ensure continuous „traction“.

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